



How to import the
Leone_XCN library in **Dental Wings**[®]
CAD software

1. Open the DW Client App

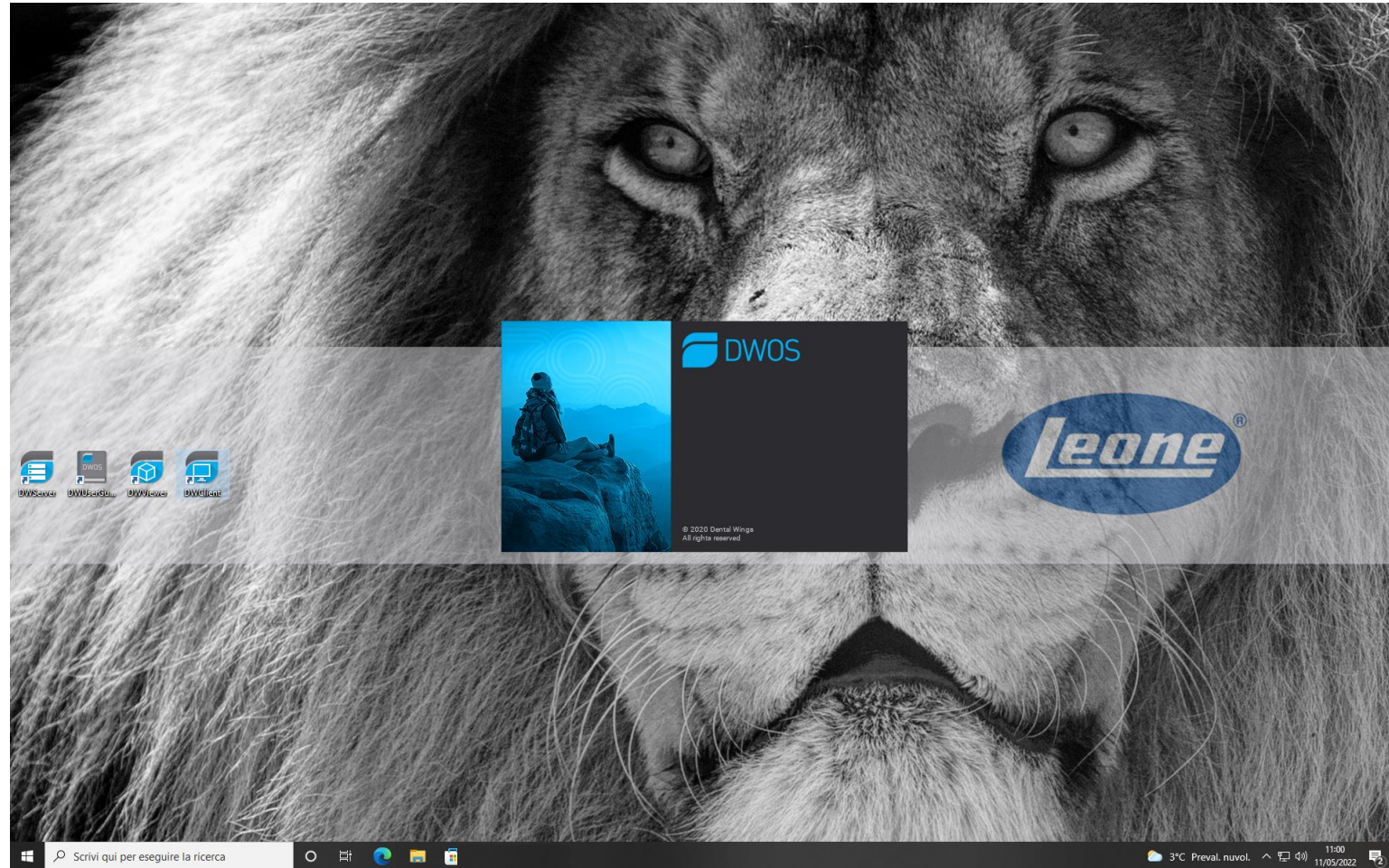


How to import the
Leone_XCN library
in **Dental Wings®**
CAD software


2. Select «Application»

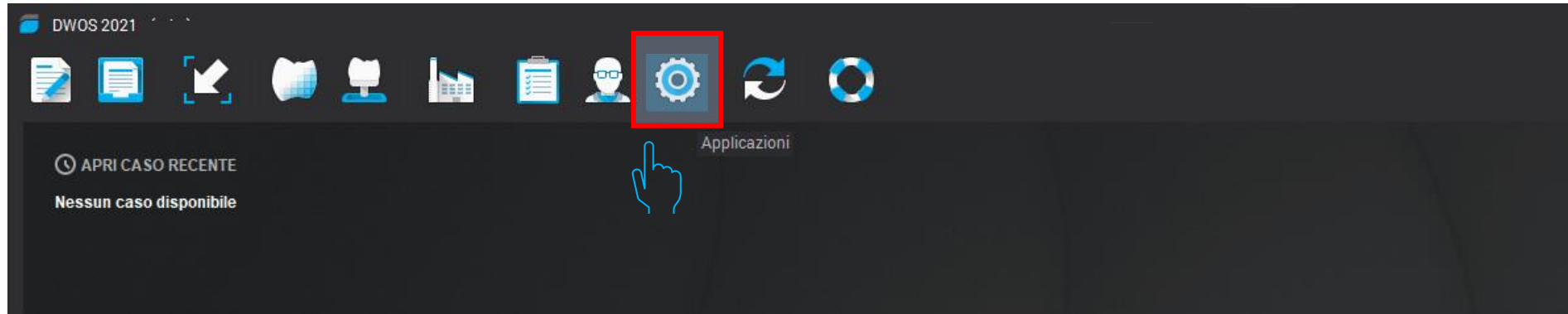


3. Select «Implant Kit»

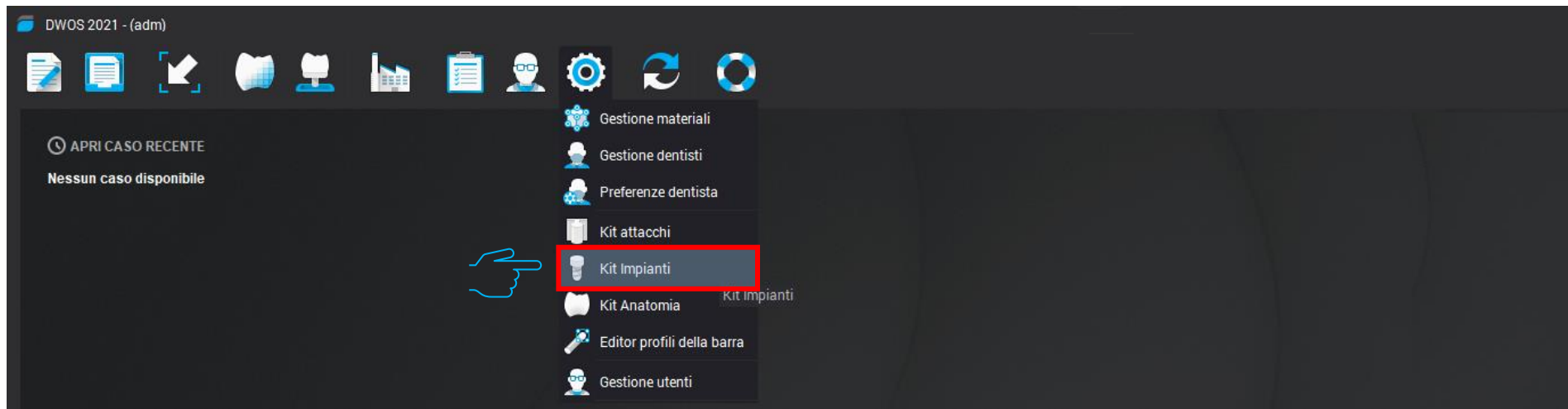


4. In the list menu of Implant Kit select «**Import Implant Manufacturer**»

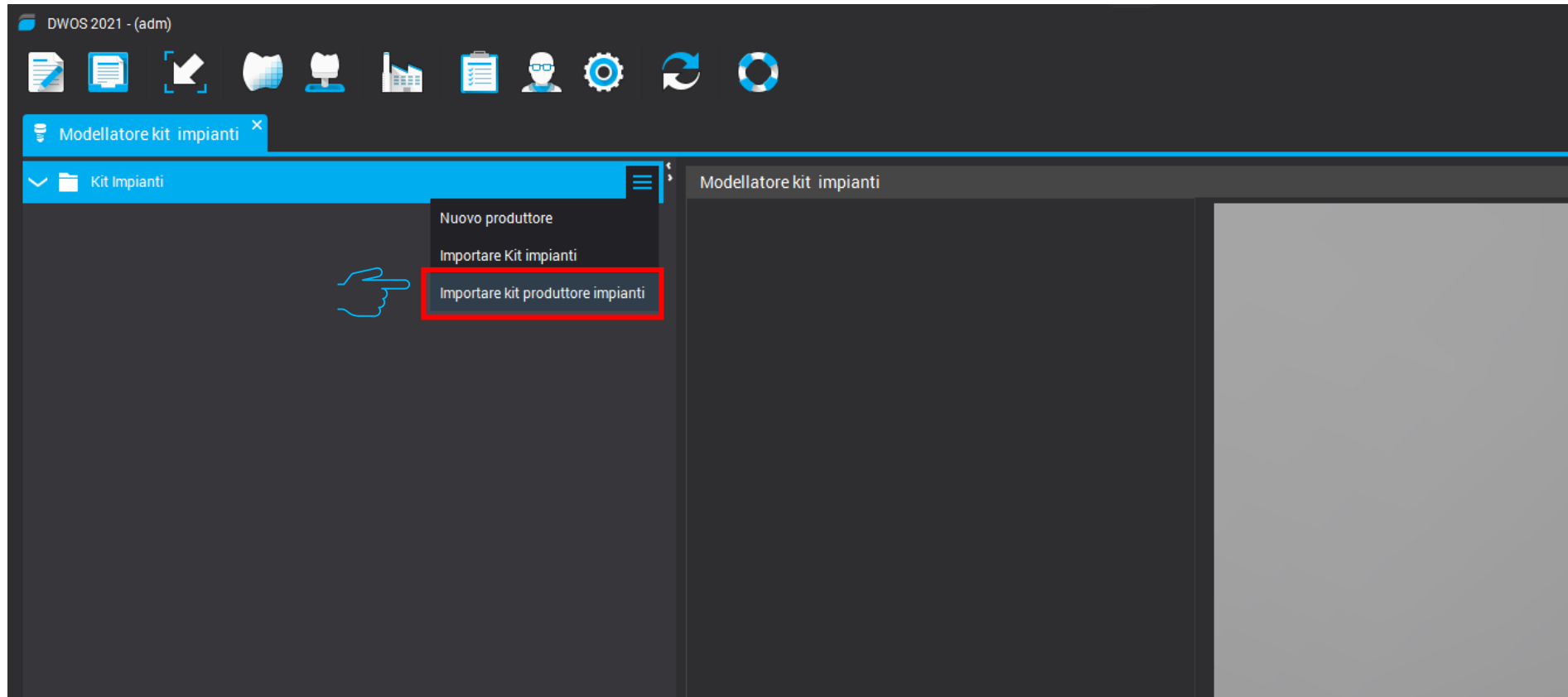

5. Select the zipped folder «**Leone-XCN-AAAA.zip**»
That you can download directly from Leone website
<https://www.leone.it/english/implantology/>

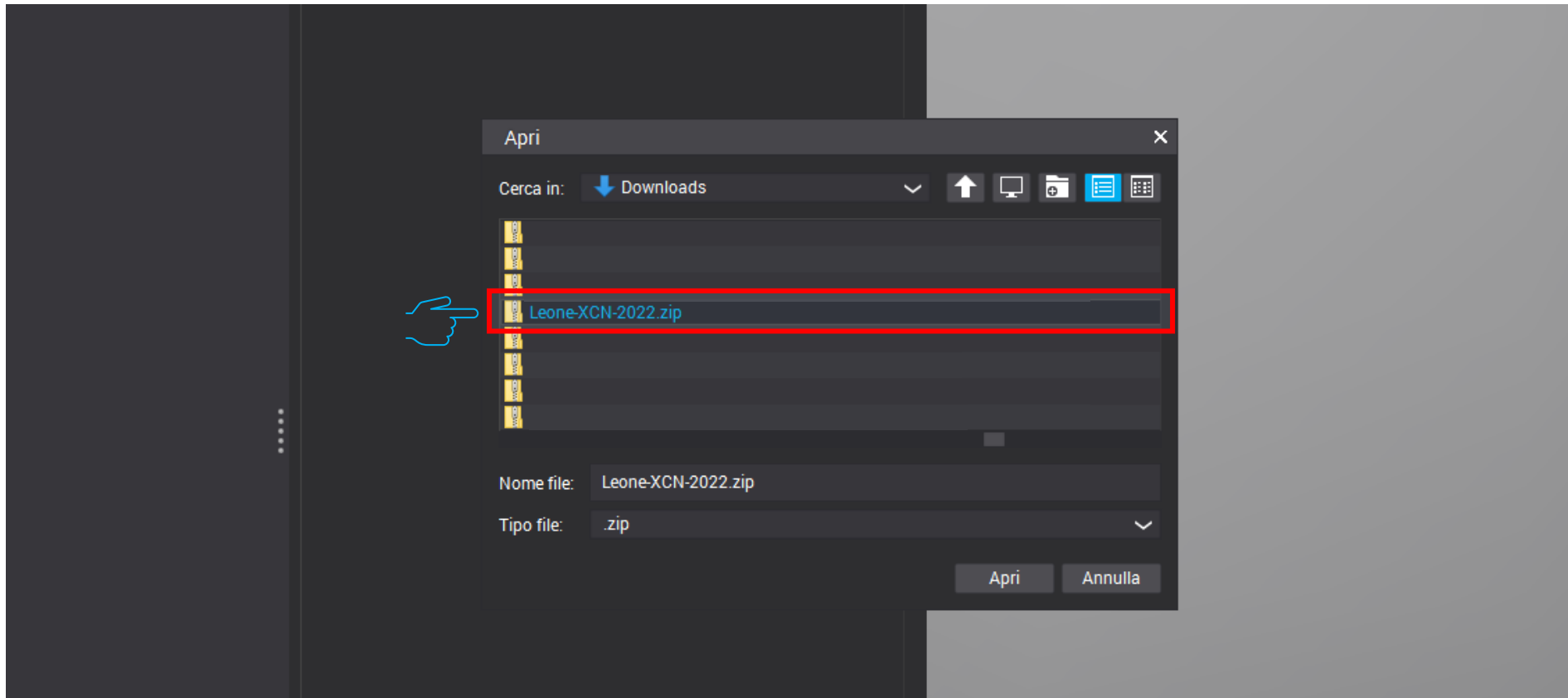

5. At the end of the procedure the Leone libraries will be available into the software

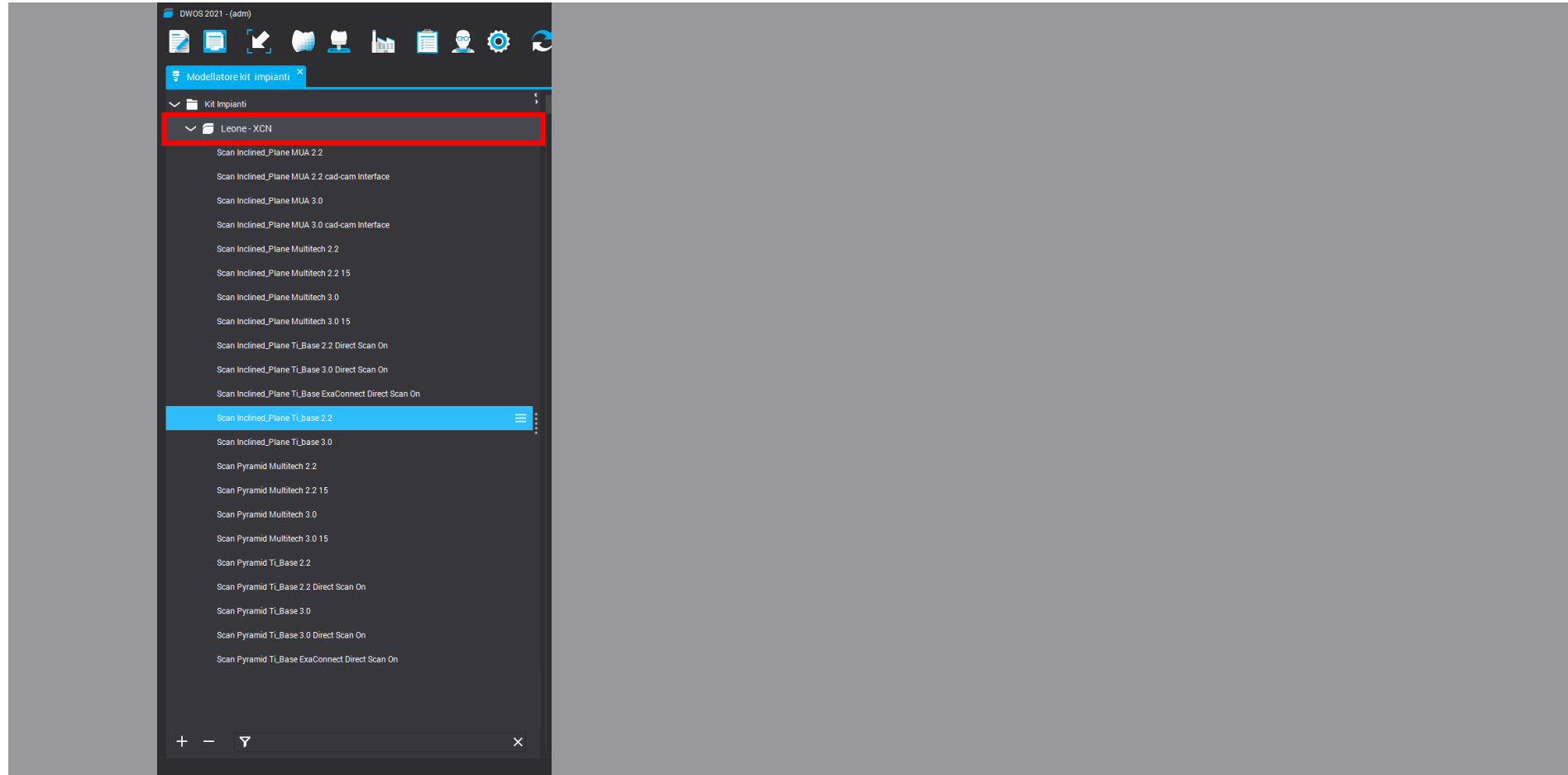

How to import the
Leone_XCN library
in **Dental Wings®**
CAD software