

# MEMORIA® LEAF SPRING ACTIVATED EXPANDER (PATENTED)



EXPANDER ACTIVATION GUIDELINES	LEAF expander cementation		FIRST visit		SECOND visit		THIRD visit
<b>6 mm</b> <b>A2703-06 - A2704-06</b>	spring activation 0 turns	after 6 weeks	10 turns	after 4 weeks	10 turns	after 4 weeks	10 turns
<b>9 mm</b> <b>A2703-09 - A2704-09</b>	spring activation 0 turns	after 8 weeks	15 turns	after 6 weeks	15 turns	after 6 weeks	15 turns